

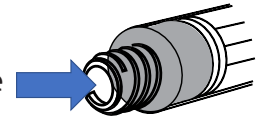
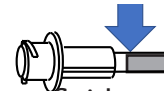
# ENFit® Cleaning Procedures

## Feeding Tubes with Male ENFit Connectors


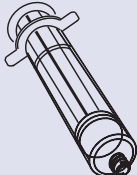


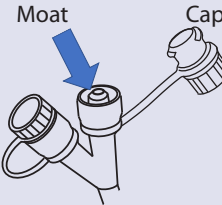
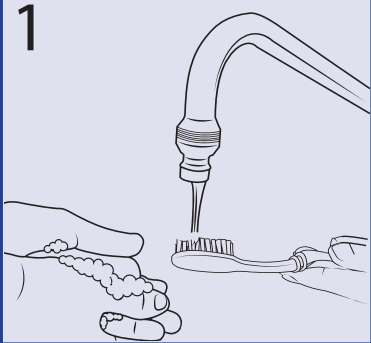
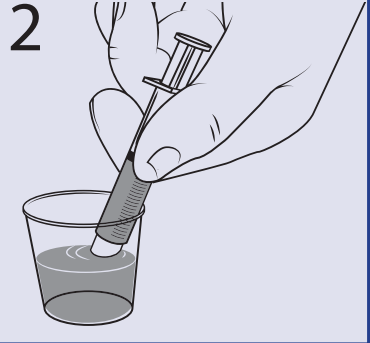
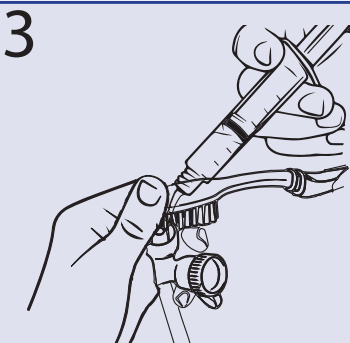
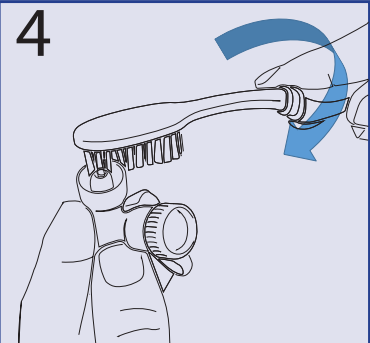
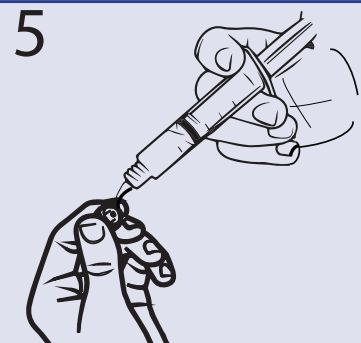
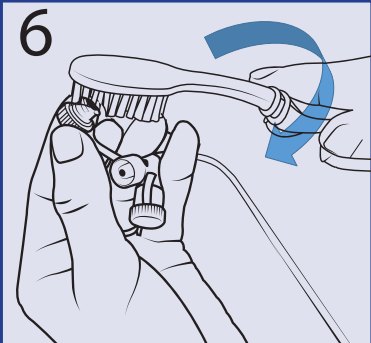
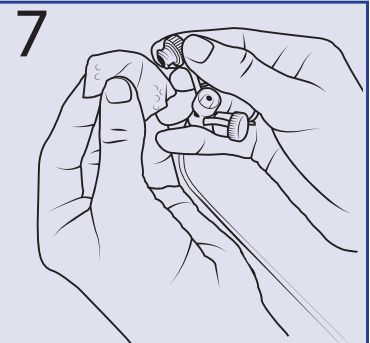
(e.g. Nasogastric, Transpyloric, Orogastric, Percutaneous Endoscopic Gastrostomy Tubes and other ENFit devices)

**Tips for keeping ENFit feeding tube ports clean. Inspect before you connect!**

- **Priming Feeding Sets** - Stop priming before fluid reaches the end of the tube.
- **ENFit Syringe Draw Up** - Wipe medication and nutrition from tip/outer threads, keep fluids out of dead space before connecting to feeding tube.



For best results, follow these instructions to clean tubes at least once a day or whenever material is visible.

Tube Cleaning Supplies & Terms				
 Cup of sterile or clean tap water	 Syringe	 Gauze	 Brush* or ENFit specific cleaning tool	 ENFit Feeding Tube
<b>Note:</b> Use a disposable brush or follow manufacturer's instructions if using ENFit specific cleaning brush.				
<b>1</b> 		<b>2</b> 		
<b>3</b> 		<b>4</b> 		
<b>5</b> 		<b>6</b> 		
<b>7</b> 				

**Repeat steps 3 through 6 until cap and tube are thoroughly clean.**

\* A manual toothbrush is regulated as a medical device intended to remove debris from the teeth in some jurisdictions. Consult your licensed healthcare provider or Risk Manager regarding recommended use for cleaning feeding tube ports. Dispose of single use devices as instructed. Cleaning procedures courtesy of Children's Mercy Kansas City.  
© GEDSA 2018 ENFit is a registered trademark of GEDSA.